

What Do My Wash Screens Mean? - SmartBand



Desiree Garcia
21 days ago Updated

Follow

Get Familiar with your 'Wash Screens'

In this section, we will go over the different types of wash screens you should expect and what they mean when you are wearing your SmartBand. Let's get started!

Room Entry Reminder



This screen will appear if you forgot to wash your hands before entering the patient's room. This is simply a reminder letting you know that you have 30-60 seconds (depending on your system settings) to wash your hands and remain compliant.

Room Exit Reminder



If you have been in the patient room longer than 60 seconds and you exit without a wash, you will receive a 'Wash Hand Exit'. This screen lets you know that you have 60 seconds to wash your hands to remain compliant.

Wash Timer



The wash timer indicates that you are washing your hands. Once you begin the hand washing motion, you will see a purple screen on your SmartBand that says 'Wash Timer'. The timer will begin to calculate your wash time.

Coaching Screens:

*The following screens **DO NOT** affect your compliance score nor your data. They serve as a coaching tool to help with your washing duration average.*

Wash Too Short Reminder



If your wash is not long enough to reach your facilities wash duration setting, this screen will appear. While not meeting the duration setting is not ideal, this is a coaching tool to help improve duration and does not affect your compliance.

Positive Wash Reminder(s)



When you've completed a wash and the duration meets your facility's wash duration setting, you will see this screen. This screen shows that you have reached the goal for the wash, well done!

Contact Support

For continued help and support, please contact us at help@vitalacy.com 😊